

PM4150 Backpack / Carry Bag Instructions

Part #507303

Placing the Portable Oxygen Concentrator (POC) in Carry Bag

• Before placing POC inside carry bag, make sure bag is clean and empty

• When the POC is properly inserted into carry bag:

- 1) The POC top will be even with the top of the bag, as shown.
- 2) The control panel can be seen thru the clear window, as shown.

Correct



Incorrect



Possible Problems

- Internal bottom tray backwards
- Something in Bag

Properly inserted Plastic Tray



Shoulder Strap Configuration



Backpack straps (2) are placed inside pouch at bottom of bag when not in use, as shown above.



Backpack Configuration

All four (4) straps are used with backpack.



www.oxygenportableusa.com

Tel: 1.714.989.6389